

Miru

1day Menicon Flat Pack



Good start. Good day.







The world's first flat pack daily disposable contact lens.

Comfortable, clear vision

Miru 1day Flat Pack lenses are made from poly (HEMA-GMA), a material that bonds readily with water and resists drying for all-day hydration and comfort.

With spherical lenses, spherical aberration can cause blurriness because the focal point of light rays passing through the lens varies according to the ray's distance from the lens center.

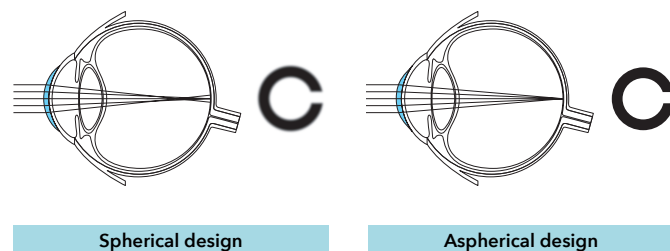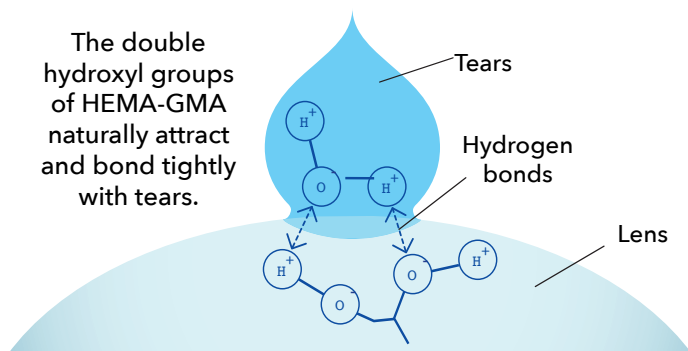


With the Miru 1day Flat Pack aspherical lens design, spherical aberration is minimized²; light rays come together at the correct focal point to help promote a sharp, clear view, especially in low light.

SMART TOUCH™

Miru 1day Flat Pack was specifically developed to reduce the chance of lens inner surface contamination. When opened, the outer surface of the lens is always facing up, ready to be picked up.

This eliminates confusion about lens orientation and reduces the chance of touching the inner surface of the lens, making it less likely that skin oil or harmful microorganisms will get trapped between the lens and the wearer's eyes.

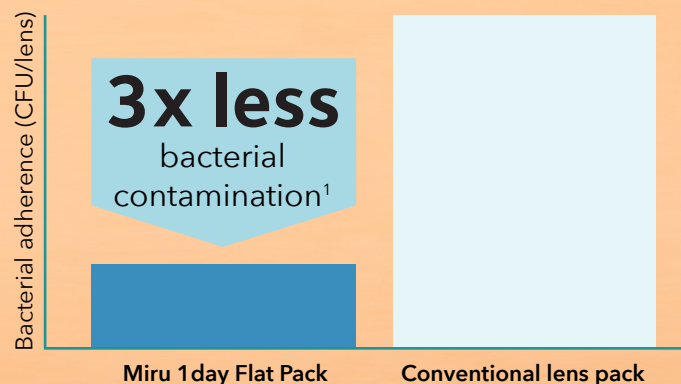

The number of bacterial colonies isolated from lenses after handling to remove from packages.

Environmental focus

Miru 1day Flat Pack uses less.

Miru 1day Flat Pack is **8x thinner** than conventional blister packs.



Conventional lens pack

Miru 1day Flat Pack

A one year supply of **Miru 1day Flat Pack** (720 lenses) generates roughly 80% less waste than conventional blister packaging.³



Conventional lens pack

Miru 1day Flat Pack


^{1,2,3}Menicon data on file.



Product Specifications


Sphere

Characteristics	Material	hioxifilcon A
	Water Content	57%
	Dk/t @ -3.00D	25.38 x 10 ⁻¹¹ (cm ² /sec) • (mLO ₂ /(mL x mmHg))
	Center Thickness	0.10 mm @ -3.00D

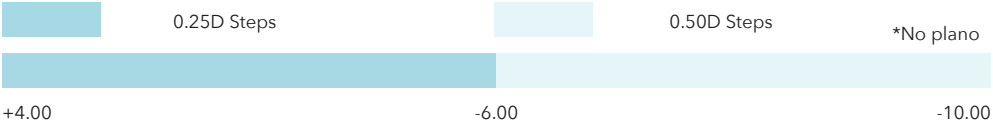Parameters	Diameter	14.20mm
	Base Curve	8.4mm/8.6mm
	Sphere	+4.00D to -10.00D
		

MIRU 1day Menicon Flat Pack	Handling Tint	Light blue
	Manufacturing	CENTRAFORM™ Process
	Wear	Daily wear, daily replacement
	Packaging	30 & 90 Lens pack

Easy-to-open packaging

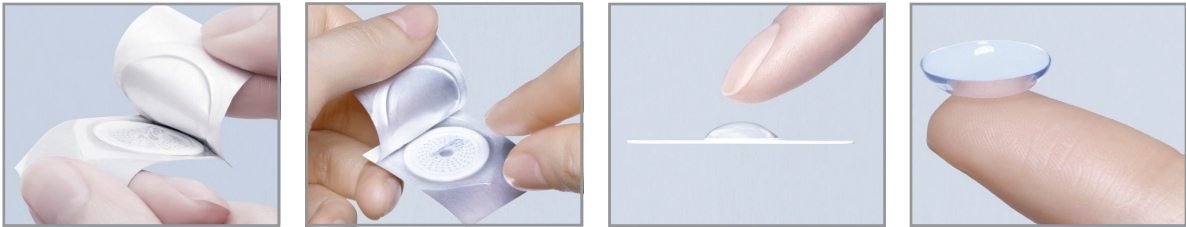

www.meniconamerica.com

Manufactured by Menicon Co. Ltd. See instructions for use in the Package Insert.
©2024 Menicon Co. Ltd. US-MI002-3