




Miru 1 day UpSide multifocal

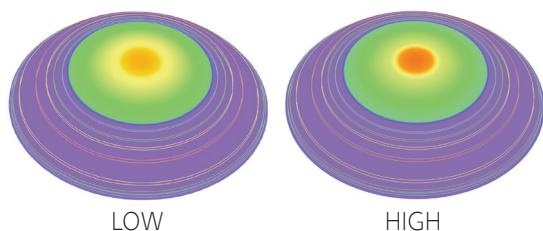

Neuro Adaptive™ Design Center near vision

Easy adaption with natural smooth transitions through near, intermediate and far viewing.

Successful fitting with Miru 1 day UpSide multifocal

Miru 1 day UpSide multifocal has been specifically designed to make multifocal fitting easier.

- Maximize the plus sphere powers during distance vision refraction
- Account for vertex distance (>+/-4D) and mean spherical equivalent for astigmatism up to 1D
- Determine dominant eye
- Select the initial add. Extra distance plus power should be offered using real vision assessments as first choice and is often well accepted.

HIGH
1st Time
Fitting
Success¹

95%
on the first lens

100%
within 2 lenses

Initial lens selection	Distance vision enhancements	Near vision enhancements
Add +0.75D to +1.25D 	Refine distance power 	Increase plus power
Add +1.50D to +1.75D 	Refine distance power 	Switch to HIGH Remove extra +0.50D from initial lens selection
Add +2.00D and above 	Refine distance power 	Increase plus power

General tip:

Always push the plus giving the least reading Add needed. For enhancements use full aperture trial lenses increasing the power as needed. Binocular adjustments are also possible.

©2024 Menicon Co. Ltd US-MI056